## [SE] Bruksanvisning

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## [EN] Instructions

### Purpose and protection

The product is intended to protect against drip contamination.

### Storage and minimum durability

Store the product in its carton. Avoid exposing the product to direct sunlight during storage. Provided that this is met, the product has a minimum shelf life of 5 years after the date of manufacture. Date of manufacture is indicated on the visor label.

### Use

- Remove the protective film on both sides of the visor before use.
- Open the velcro straps.
- Place the visor against the forehead and fasten the visor around the head by closing the velcro straps
- Removed in reverse order.

### Disinfection and cleaning

The product is not sterile. Clean with:

- Water: Lukewarm water with the addition of a neutral detergent. Rinse afterwards with clean water and dry with a soft sponge, wool cloth or similar.
- Disinfection: Surface disinfection (Ethanol 50–80%, isopropanol 5–10% with surfactant)

Never use sharp objects to close or dirt.

### Storage between uses

The product is sensitive to scratches after the protective film has been removed. If possible, avoid direct sunlight as UV light shortens the life of the product.

### Damage, maintenance and service

Scratched or damaged product should be discarded. Otherwise, the product requires no maintenance.

### Reuse

The product is approved to clean and reuse. Under the right conditions, the product's service life is several months, even with frequent use. Depending on handling, the service life can be both shorter and longer. When the product loses its transparency or feels unhygienic, it should always be discarded. It is always up to the user to decide when to dispose of it.

### Recycling

Sorted as plastic packaging. Contaminated product must be sorted according to the employer's instructions.

### Manufacturer

Packing Kungsör AB SE556705385401  
Fabriksgatan 5 info@packing.se  
736 36 Kungsör +46 (0) 227-312 55  
Sweden

### Certification


CE-approved. Tested according to EN 166:2001  
The EU type control is carried out by RISE Research Institutes of Sweden AB  
Box 857, SE-50115 BORÅS. Notified No. 0402

### Allergy

The product may cause an allergic reaction in susceptible persons.

### Marking

ART11534 1 S -EN 166 3 S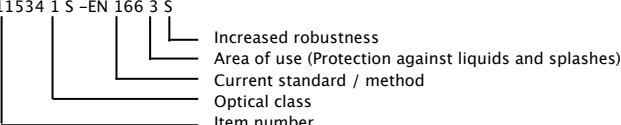

- Increased robustness
- Area of use (Protection against liquids and splashes)
- Current standard / method
- Optical class
- Item number

**This manual should be made free of charge for each person.**

The EU declaration of conformity can be downloaded at this address:  
[https://www.packing.se/skyddsvisir/eu\\_forsakran.pdf](https://www.packing.se/skyddsvisir/eu_forsakran.pdf)